



ACURATE *neo*[™]

Aortic Valve System

SAVI-TF POST-MARKET REGISTRY

Real-world Experience Using the ACURATE *neo* Prosthesis: 30-day Outcomes of 1,000 Patients Enrolled in the SAVI-TF Registry

Möller H, et al. *EuroIntervention*. 2018;13:e1764-e1770.

One-year Outcomes of the European Post-market Registry Using the ACURATE *neo* Transcatheter Heart Valve Under Real-world Conditions in 1,000 Patients

Kim WK, et al. *JACC: Cardiovasc Interv*. 2018;11(14):1368-74.

Study Design


European, single-arm, multicenter, all-comers registry, conducted at 25 centers, evaluating 1,000 patients following CE-marking of ACURATE *neo* between October 2014 and April 2016.

The SAVI-TF Post-market Registry was initiated to evaluate the safety and performance of the ACURATE *neo* Aortic Valve System in a large patient population treated according to the European clinical practice.


Clinical Highlights

High Procedural Efficiency

ACURATE *neo* showed high procedural success, low rates of complications, and short procedure times:

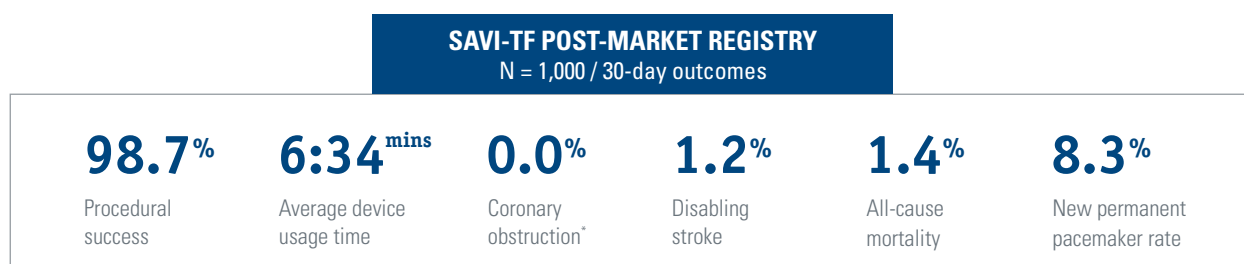

*Requiring reintervention

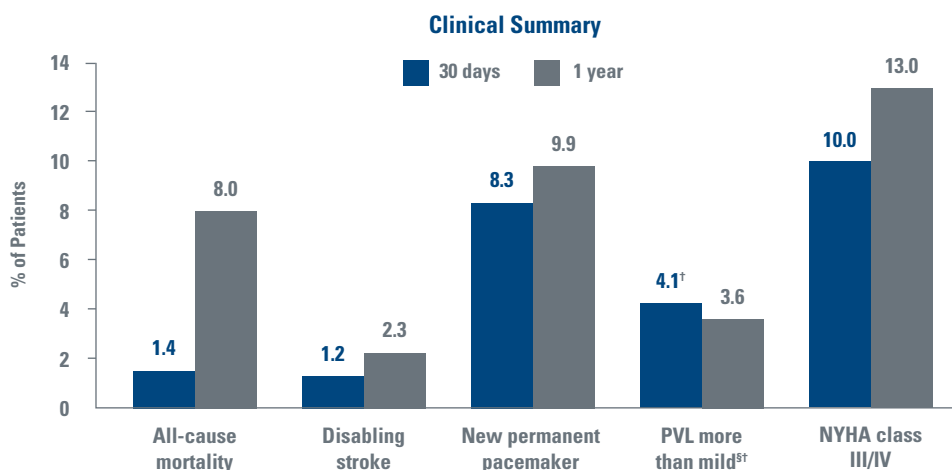


[§] PVL more than mild was observed only in moderately and severely calcified annuli

[†] At discharge/7 days

Excellent Hemodynamic Performance

Large effective orifice areas and low transvalvular gradients at discharge and sustained at 1 year

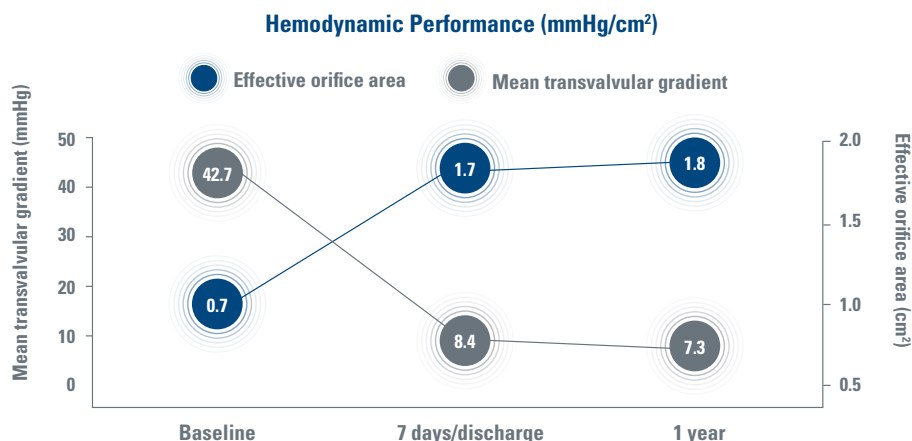

Key Results

Echocardiographic Parameters	7 days/discharge	1 year
Effective orifice area (cm ²)	1.77 ± 0.46 (n = 416)	1.84 ± 0.43 (n = 257)
Mean aortic gradient (mmHg)	8.4 ± 4.0 (n = 807)	7.3 ± 3.7 (n = 484)
Aortic regurgitation	n = 844	n = 587
≤ Grade 1 (none to mild)	95.9%	96.4%
Grade 2 (moderate)	4.1%	3.4%
Grade 3 (moderate to severe)	0.0%	0.2%
Grade 4 (severe)	0.0%	0.0%

Clinical Outcomes	30 days/n = 998 ^a	1 year/n = 983
Early safety^b	8.6%	
Mortality	1.4%	8.0%
cardiovascular	1.0%	3.5%
Stroke	1.9%	3.5%
disabling	1.2%	2.3%
Life-threatening bleeding	1.3%	2.0%
Major bleeding	4.4%	
Acute kidney injury stage 2 or 3	1.3%	
Major vascular complication	3.2%	
Coronary obstruction requiring reintervention	0.0%	0.0%
Repeat procedure	0.0%	0.5%
Myocardial infarction	0.3%	1.3%
Endocarditis	0.0%	0.8%
Valve thrombosis	0.0%	0.0%
Cardiac tamponade	0.4%	
New pacemaker implantation	8.3%	9.9%

a) One patient withdrew consent after treatment; one did not return for 30-day visit.

b) A composite of all-cause mortality, all stroke, life-threatening bleeding, coronary obstruction requiring intervention, major vascular complication, acute kidney injury stage 2 or 3, and repeat procedure for valve-related dysfunction.

All trademarks are the property of their respective owners. CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings, and instructions for use can be found in the product labeling supplied with each device. Information for use only in countries with applicable health authority product registrations. Not intended for use or distribution in France, Japan, and the USA.

SH-647903-AA Printed in Germany by medicalvision

Boston Scientific

Advancing science for life™

www.bostonscientific.eu/ACURATEneo

© 2019 Boston Scientific Corporation or its affiliates. All rights reserved.
DINSH0235EA